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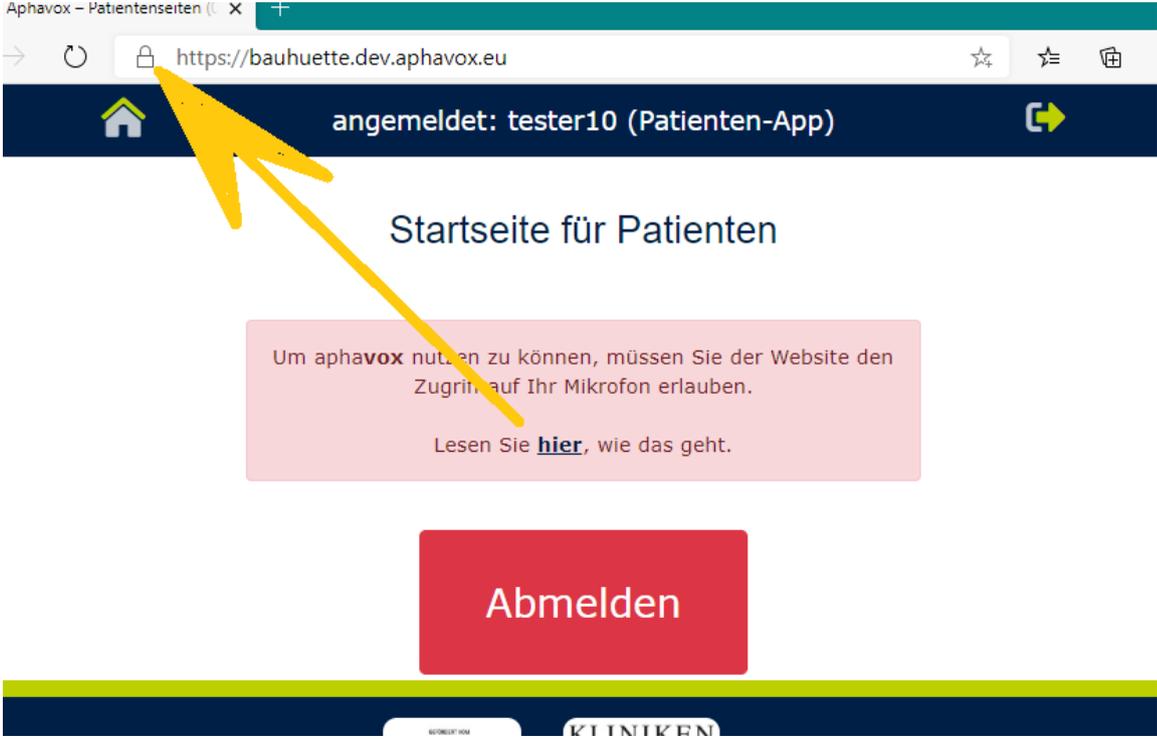
The screenshot shows a web browser window with the URL `https://bauhuette.dev.aphavox.eu`. The browser's address bar contains a lock icon, a refresh icon, and navigation icons. Below the address bar is a dark blue navigation bar with a home icon on the left, the text "angemeldet: tester10 (Patienten-App)" in the center, and a right-pointing arrow icon on the right. The main content area has a white background with the heading "Startseite für Patienten". A pink rectangular box contains the text: "Um aphavox nutzen zu können, müssen Sie der Website den Zugriff auf Ihr Mikrofon erlauben. Lesen Sie [hier](#), wie das geht." Below this box is a large red button with the text "Abmelden". At the bottom of the page, there is a dark blue footer with a yellow bar above it, containing the text "KLINIKEN" and "KONFERENZ". A yellow arrow points from the lock icon in the address bar to the pink permission box.

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The image shows a browser window with the address bar displaying "https://bauhuetten.dev.aphavox.eu". A security notification is visible, stating "Verbindung ist sicher." (Connection is secure). Below this, a settings menu is open, listing various site permissions. A yellow arrow points to the "Websiteberechtigungen" (Website permissions) option. The menu also shows "Tracking-Verhinderung" (Tracking prevention) and a list of trackers for the current site, which is currently set to "Ein" (On).

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